



CDI's Ink Aid™

- Description-

CDI's Tissue Marking Dyes are formulated not to require an additional mordant. However, Ink Aid™ can improve adherence with certain tissue and can be helpful particularly with fatty tissue.

- Procedure -

Spray onto inked tissue (1-2 sprays) to help set/bond the dye to tissue. For optimal results, pat tissue dry prior to applying dyes. Only small amounts of dyes are necessary for effective marking.

- Precautions -

If In Eyes	Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue Rinsing. If eye irritation persists: Get medical attention.
If On Skin	Wash with plenty of water. If skin irritation or rash occurs: Get medical advice/attention.

Color	Pale Yellow
Manufacturer	Cancer Diagnostics, Inc.
Intended Use	Laboratory
Storage Requirements	Room Temp (15°-30° C)
D.O.T. Regulated?	No
I.A.T.A. Regulated?	No
Biodegradable?	Yes
CE Marked?	Yes
Emergency Contact (USA)	800-424-9300
Emergency Contact (Intl.)	1-352-323-3500

