

DANGER:

Toxic if inhaled. Causes severe skin burns and serious eye damage. May cause allergy or asthma symptoms or breathing difficulties if inhaled. May cause an allergic skin reaction. May cause genetic defects. May cause cancer. May cause damage to organs (lungs, nose) through single, prolonged, or repeated exposure.



Deionized Water	7732-18-5
Formic Acid	64-18-8
Formaldehyde	50-00-0



For Invitro Diagnostic Use Only

DIRECTIONS FOR USE:

Calfor is formulated to fix and decalcify simultaneously. Fresh tissue can be placed directly into Calfor; Needle biopsies may be completely fixed and decalcified in one hour or less. Small bone specimens and calcified arteries may be completely fixed and decalcified in 2-4 hours. An excellent decalcifier for large dense specimens for fixation and to begin decalcification. Calfor is a proven formulation for ISH and IHC. Use only outdoors or in a well-ventilated area. Do not breathe dusts, mists, or vapors. Wash body thoroughly after handling. Wear protective gloves and clothing. Wear eye and face protection. Wear NIOSH approved respiratory protection. Contaminated work clothing must not be allowed out of the workplace. Obtain special instructions before use. Do not handle until all safety precautions have been read and understood. Do not eat, drink, or smoke when using this product. **IF INHALED:** If breathing is difficult, remove person to fresh air and keep comfortable for breathing. If experiencing respiratory symptoms: Call a doctor. Call a doctor. Specific treatment, see first aid section on SDS. **IF SWALLOWED:** Rinse mouth. Do NOT induce vomiting. **IF ON SKIN (OR HAIR):** Rinse skin with water/shower. Take off immediately all contaminated clothing and wash it before reuse. Immediately call a doctor. If skin irritation or rash occurs: Get medical attention. If exposed or concerned: Get medical attention. If exposed or concerned: Call a doctor. Call a doctor if you feel unwell. **IF IN EYES:** Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue Rinsing. **STORAGE:** Store in a well-ventilated place. Keep container tightly closed. Store locked up. **DISPOSAL:** Dispose of contents/container in accordance with local regulations. **NOTICE:** Read safety data sheet for additional information.

ANTIDOTE

External: Flood with water for at least 15 minutes.

Internal: Raw egg white, milk, rice gruel. Call a physician.

DANGER: Contains dilute acid. May be corrosive to metallic parts, skin, eyes and clothing. Avoid prolonged contact.

EC REP mdi Europa GmbH Langenhagener Str. 71 D-30855 Langenhagen Germany +49-511-3908 9530

Cancer Diagnostics, Inc. 4300 Emperor Blvd., 400 Durham, NC 27703 (877) 846 5393 Fax: (877) 817 1716

REF CFQ038 Gal (3.8L) CFQ025 Gal (3.8L) Cube CFQ100 1000mL CFQ438 Gal (3.8L), CS/4 CGQ400 1000mL, CS/4

Fixative Decalcifier
CalFor[®]