



Fusion Micro Easy Attacher

Step 1. Place your Compatible* Microtome Blade into the channel with the sharp edge to the left.



Step 2. Close and hold the lid down, and using firm pressure, push the handle into the opening. Open the lid, if the blade is not in the slit of the handle, repeat this step.

(Line up the Aperture on handle, and on the blade, if it is not already)





Fusion Micro Easy Attacher

Step 3. Twist off the “Nib” from the back of the handle, and place it in the aligned aperture. Close top of Easy Attacher and press down firmly to secure nib. Remove Fusion Micro carefully. Single-Use, dispose in sharps container when dull. **Do not re-use.**



*Compatible Microtome Blades:

-LPJ3580 - Bevel® HD, Low Profile Microtome Blades BX/50

-LPJ3581 - Bevel® PLUS, Low Profile Microtome Blades BX/50