



How to use Fusion Micro Handles IFU

Contents: FUS999 Combo Pack (50 Handles/ 50 blades)



SOLD SEPARATELY

STEP 1: DISPENSE BLADE AND PLACE INTO EZ ATTACHER (SHARP SIDE TOWARD EDGE)





How to use Fusion Micro Handles IFU

STEP 2: TWIST OFF "NIB" FROM BACK OF HANDLE. GENTLY INSERT BLADE INTO SLIT OPENING IN HANDLE. ALIGN HOLE IN HANDLE WITH HOLE IN BLADE AND PLACE NIB INTO HOLE ON HANDLE. (IT IS NOT NECESSARY TO FORCEFULLY PUSH NIB IN YET)





How to use Fusion Micro Handles IFU

STEP 3: CLOSE EZ ATTACHER'S HINGED LID ONTO NIB WHICH WILL SNAP NIB INTO PLACE AND SECURE BLADE TO HANDLE. DO NOT TRY TO REMOVE NIB. USE A NEW BLADE AND HANDLE WHEN DULL





How to use Fusion Micro Handles IFU

INFOGRAPHIC:

