



How to use Fusion Micro Handles

1. Locate the slit on the bottom of the head, away from the “Nib” on the tail end.



2. Slide the blade (Carefully!) through this slit until the aperture on the blade, and handle, line up.



3. Break off the “Nib” on the tail end.



**Cancer Diagnostics Always Recommends Using
the Easy Attacher #FUS888**



How to use Fusion Micro Handles

4. Insert this “Nib” in between the aligned apertures. Push using firm pressure until only 0-0.75mm of it remains protruding, though flush (0mm) is ideal.



Note: If more than 1 or 2 mm are left protruding, the blade is **NOT SECURE!** The apertures are most likely not aligned precisely.



**Cancer Diagnostics Always Recommends Using
the Easy Attacher #FUS888**