



Ribbon™ Tissue Section Adhesive

- General Information -

Ribbon™ is a liquid adhesive designed to promote adherence of tissue sections to glass slides. Simply added to the water bath, it enhances adhesion of paraffin sections to the glass surface of slides. **Ribbon™** will not cause background staining when using hematoxylin stains. **Ribbon™** is specially formulated to be used on glass slides without pretreatment and for use with microwave drying of sections on slides. It also works well with traditional methods of drying tissue sections in 60°C ovens. Diluted **Ribbon™** can also be used to coat slides for frozen sections and smear preparations.

- Procedure -

1. Add 4-10mL of **Ribbon™** to 1000mL of water, gently agitate.
2. Add water to water bath, use as normal.
3. Replace once per day, or as you refill water bath. No special handling is needed.

- Packaging -

Catalog#	Volume
P1000RD	500mL
PCM1005	1 Gallon