



CF2™

CYTOLOGY FIXING FLUID

- #FX1090 1 GAL (3.8L)
- #FX1090-WP 1 GAL (3.8L), DISPENSER PUMP
- #CX1090 5 GAL (19L) CUBE

For In Vitro Diagnostics Use

Date Received: _____ Date Opened: _____

Indications: CF2™ is a fixative formulated to produce improved fixation and staining of body fluids and cytospin preparations. It combines the coagulant fixative qualities of alcohol with other fixation enhancements to produce better nuclear and cytoplasmic fixation. Erythrocytes are partially lysed and do not obscure the nucleated cells of interest. Nuclear and cytoplasmic boundaries are precise and well defined.



DANGER

HAZARD STATEMENTS

Causes skin and serious eye irritation. Causes damage to organs (respiratory system). May cause damage to organs (respiratory system, central nervous system, liver, blood) through prolonged or repeated exposure. Flammable liquid and vapor.

PRECAUTIONARY STATEMENTS - PREVENTION

Wash body thoroughly after handling. Wear protective gloves. Wear eye and face protection. Do not breathe dust, vapors. Do not eat, drink, or smoke when using this product. Keep away from heat sources and open flame. No smoking. Keep container tightly closed. Ground/bond container and receiving equipment. Use explosion-proof electrical, ventilating, lighting and equipment. Use only non-sparking tools. Take precautionary measures against static discharge.

PRECAUTIONARY STATEMENTS - RESPONSE

IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue Rinsing. If eye irritation persists: get medical attention. If exposed or concerned: call a doctor. Call a doctor if you feel unwell.

IF ON SKIN: Wash with plenty of water. Specific treatment, see first aid section on SDS. If skin irritation or rash occurs: get medical attention. Take off all contaminated clothing and wash it before reuse.

IN CASE OF FIRE: Use CO₂, dry chemical, or foam for extinction.

STORAGE & DISPOSAL

Store locked up. Store in a well-ventilated place. Keep cool. Dispose of contents/container to an approved waste disposal plant.

NOTICE: Read Safety Data Sheet for additional information.

EMERGENCY CONTACT: Chemtrec 1-800-424-9300



CANCER
DIAGNOSTICS INC.

4300 Emperor Blvd - 400 Durham, NC 27703

Tel: 1-877-846-5393 Fax: 1-877-817-1716

www.cancerdiagnostics.com

Made in USA