
Trimming Knife Handle, Short Trimming Knife Handle, Long

- How to set the blade
- Anbringen der Klinge
- Mise en place de la lame
- Instrucciones para colocar la cuchilla
- Installazione della lama
- Como inserir a lâmina
- Bevestigen van het lemmet
- Use care when handling blades.
- Vorsicht beim Umgang mit den Klingen.
- Utiliser les lames avec précaution.
- Usar las cuchillas con cuidado.
- Maneggiare le lame con cura.
- Ter todo o cuidado ao manusear lâminas.
- Voorzichtig bij het gebruik van de messen.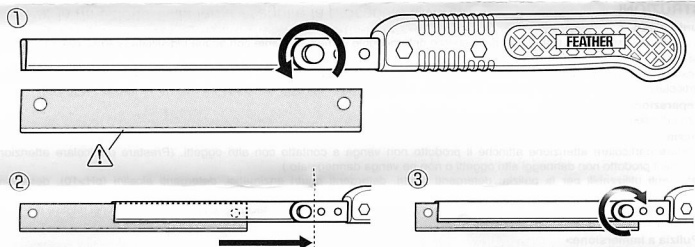

- Do in reverse order when detaching the blade.
- Zum Ausbau der Klinge in umgekehrter Reihenfolge vorgehen.
- Procédez dans l'ordre inverse pour démonter la lame.
- Para sacar la cuchilla, realice los mismos pasos en el orden inverso.
- Per la rimozione della lama seguire la procedura inversa.
- Ao desmontar a lâmina, proceda pela ordem inversa.
- Om het lemmet los te maken, omgekeerd te werk gaan.

ACCESSORIES

The following Accu-Edge Blades are currently available for the Trimming Knife Handle.
4785 Trimming blades straight edge, 130mm 4798 Trimming blades round edge, 130mm
4789 Trimming blades, 260mm

CAUTION

- Always check the lock screw is connecting to the hole of the replacement blade before use.
- Make sure the blade is fixed on the handle before use.
- Use only as a trimming instrument.
- Do not use when there is something wrong with the product.
- For prevention of contamination, dispose properly after use.

DEVICE DESCRIPTION

The Trimming Knife Handle is a reusable handle made from stainless steel and polysulfone and used with Trimming blades which are specially designed for this product.

RESTERILIZATION INSTRUCTION

WARNINGS

- Make sure to check if the device is in proper condition for reuse after reprocessing.
- Do not clean the handle with blade.
- When cleaning, do not use any organic solvents such as xylene or thinner.

LIMITATIONS ON REPROCESSING

- In order to maintain the usefulness of the device, clean and sterilize the device immediately after use following the reprocessing instructions, and store properly until next use.
- The product life depends on the status and frequency in which the product is being used.
- The device must be prepared and handled only by personnel who possess the requisite technical expertise and training to do so.

INSTRUCTIONS

Point of use

Wash off debris on the device with purified water (<40°C/104°F) immediately after use.

Transportation

Make sure to prevent the product from damaging anything, or being damaged by any other materials during transportation.

Preparation for Cleaning

Remove the blade from the handle before cleaning.

Cleaning

Make sure to prevent the product from damaging anything, or being damaged by any other materials.

Usable detergents: Neutral detergent, Neutral enzyme detergent, Alkaline detergent (pH>10), Enzyme alkaline detergent (pH>10)

Preparation of detergent: Follow the instructions of the detergents, dilute to specified concentration.

Temperature: 40°C~50°C/104°F~122°F

<Immersion cleaning>

1. Immerse the product in detergent for 15 to 20 minutes. If the device is tainted badly, increase the immersion time.
2. Rinse the device with purified water to remove detergent thoroughly.

<Immersion cleaning with the Ultrasonic bath>

1. Put detergent into the ultrasonic bath.
2. Immerse in the ultrasonic bath and process in 28 – 40kHz for 3 minutes.
3. Rinse the device with purified water to remove detergent thoroughly.

<Washer-Disinfector>

1. Keep the device in the dedicated case, put the device on an instrument tray.
2. Put instrument tray into the washer-disinfector and start the cycle.

Drying

Dry the product thoroughly under clean conditions.

Packaging

Pack the product into a sterilization container or appropriate sterilization bag.

Sterilization

Sterilize the device with autoclave.

Recommended parameters for sterilization: 134-135°C/273-275°F, 3-5 minutes

<In case of cleaning for Crutchfield-Jacob/BSE contaminated instruments.>

Recommended parameter for deactivation of prion: Autoclave with; *1 134-135°C/273-275°F, 8-10 minutes *2 134-135°C/273-275°F, 18 minutes

*1 Sterilization parameter in case of using washer-disinfector with alkaline detergent (90-93°C). *2 Sterilization parameter recommended by The Robert Koch Institute when an alkaline cleaning solution should not be used.

Storage

Store the device appropriately until next use following the rules each facility sets for the method and conditions of storage after sterilization.

ADDITIONAL REMARKS

The instructions provided above have been validated by the medical device manufacturer as being CAPABLE of preparing a medical device for re-use. It remains the responsibility of the processor to ensure that the processing as actually performed using equipment, materials and personnel in the processing facility achieve the desired result. This requires validation and routine monitoring of the process. Likewise any deviation by the processor from the instructions provided should be properly evaluated for effectiveness and potential adverse consequences.

 Feather Safety Razor Co., Ltd Osaka, Japan	Sakura Finetek Europe B.V. Flemingweg 10a 2408 AV Alphen aan den Rijn The Netherlands	Sakura Finetek USA, Inc 1750 W 214th Street Torrance, CA 90501 USA
Made in Japan		

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