



Nail-RX™

- General Information -

Nail-RX™ is a keratin softening agent for nail specimens. The solution softens brittle nail clippings and makes specimens easier to cut and process. Nail RX™ also helps to keep specimen from washing off the slide. Nail-RX™ **RAPID** is a stronger version of the classic formulation. Use pre-processing.

- Fixation Procedure -

1. Remove specimen from fixative and rinse gently for 1 minute in running water.
2. Place in Nail-RX™ for 1 hour
(Longer depending on specimen size).

*Nail-RX™ **RAPID** works in as little as 15 minutes.
3. Wash gently in water for 5 minutes.
4. Place cassette on processor for routine processing.

Optional: Following processing and embedding, wax block can “soak” for 5 minutes in Nail-RX™ for 5 minutes if necessary, prior to cutting by placing cut surface of specimen down in Nail-RX™.

- Packaging -

Catalog#	Volume	Product
NL0500	500mL	Nail-RX™
NLR0500	500mL	Nail-RX™ RAPID
NLR05	5 Gallon CUBE	Nail-RX™ RAPID

