

## Freeze Spray

**WARNING:** FLAMMABLE. CONTAINS GAS UNDER PRESSURE; MAY EXPLODE IF HEATED. PROTECT FROM SUNLIGHT. STORE IN A WELL VENTILATED PLACE. KEEP COOL. PRESSURIZED CONTAINER. DO NOT PIERCE OR BURN, EVEN AFTER USE. DO NOT EXPOSE TO TEMPERATURES EXCEEDING 122°F/50°C. Use in a well ventilated area. Avoid spraying in eyes. Vapor may cause frostbite on skin. Keep out of reach of children. The intentional misuse by deliberately inhaling contents may be fatal. Do not spray on an open flame or other ignition source. Use only with adequate ventilation. First Aid Inhalation: Immediately remove to fresh air. Eyes: Immediately flush with water. Skin: Immediately wash with warm water. Treat for frostbite if necessary. Call physician if necessary. See Safety Data Sheet for complete safety information.

**24 HOUR MEDICAL EMERGENCY 1-800-424-9300**

Contains: 1,1-Difluoroethane (CAS# 75-37-6)

Manufactured By:

Cancer Diagnostics, Inc.  
4300 Emperor Blvd. #400  
Durham, NC 27703  
Tel: 877-846-5393



## DIRECTIONS

Insert extension tube. Tear tab from trigger assembly. 60 Below™ rapidly cools paraffin blocks and tissue. Two quick bursts ensure deeper penetration into specimens. 60 below™ contains no CFC's, and leaves no residue. Spray 6 to 8 inches toward surface.

Read entire label before using. Hold can nozzle approximately 3" from paraffin block and tissue. Spray short blasts directly onto surface. Frost will appear on surface when cooled. Hold can upright while spraying. 60 Below™ contains a liquefied propellant which will expel as a non-cooling gas if can is held upside down, sideways, or is shaken. If sprayed for extended time, container will become very cold and the pressure will weaken. Spray short blasts or wear gloves to insulate hands. For professional use only.

SP-11516 DOT-2Q M5746



24 Hour Emergency Contact: **CHEMTREC**, 1-800-424-9300 USA and Canada

Order Toll Free: (877) 846 5393 Fax: (877) 817 1716 [www.cancerdiagnostics.com](http://www.cancerdiagnostics.com)

# 60below™

- Non-Ozone Depleting
- Rapidly Evaporating
- Zero Residue

Ozone Safe. For rapid freezing of fresh or paraffin embedded tissue.

## KEEP OUT OF REACH OF CHILDREN

WARNING: FLAMMABLE  
CONTAINS GAS UNDER PRESSURE  
MAY EXPLODE IF HEATED. READ BACK OF  
LABEL CAREFULLY.

**FOR LABORATORY USE ONLY**

10 oz (284g) #06206 PK/6  
#06212 (cs/12)

For in vitro diagnostic use only

# Freeze Spray



CANCER  
DIAGNOSTICS, INC.

Made in USA