



B-5•Sub™

MERCURY FREE FIXATIVE

#FX1058 500ML

#FX1061 1 GAL (3.8L),

#FX1062 1 GAL (3.8L), 4/CS

For In Vitro Diagnostics Use

Date Received: _____ Date Opened: _____

B-5•Sub™ is designed to fix lymphoid and hematopoietic tissues. B-5•Sub™ accomplishes fixation similar to that of B-5 without the use of mercury. It is the fixative of choice for lymph nodes, bone marrow biopsies and other lymphoid and hematopoietic tissues.



DANGER

HAZARD STATEMENTS

Fatal if inhaled. Harmful if swallowed. Causes severe skin burns and serious eye damage. May cause allergy or asthma symptoms or breathing difficulties if inhaled. May cause an allergic skin reaction. May cause genetic defects. May cause cancer. May cause damage to organs (lungs, nose) through single, prolonged, or repeated exposure.

PRECAUTIONARY STATEMENTS - PREVENTION

Do not breathe dust, mists, or vapors. Use only outdoors or in a well-ventilated area. In case of inadequate ventilation where IDLH, OSHA STEL or PEL values reached wear NIOSH approved respiratory protection. Wash body thoroughly after handling. Do not eat, drink, or smoke when using this product. Wear protective gloves and clothing. Wear eye and face protection. Contaminated work clothing must not be allowed out of the workplace. Obtain special instructions before use. Do not handle until all safety precautions have been read and understood.

PRECAUTIONARY STATEMENTS - RESPONSE

IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue Rinsing.

IF ON SKIN: Rinse skin with water/shower. Take off immediately all contaminated clothing and wash it before reuse. Specific treatment, see first aid section on SDS. If skin irritation or rash occurs: get medical attention. If exposed or concerned: get medical attention/call a doctor. Call a doctor if you feel unwell.

IF INHALED: If breathing is difficult, remove person to fresh air and keep comfortable for breathing. If experiencing respiratory symptoms: Call a doctor. Immediately call a doctor. Specific treatment is urgent, see first aid section on this SDS.

IF SWALLOWED: Call a doctor if you feel unwell. Rinse mouth. Do NOT induce vomiting.

IN CASE OF FIRE: Use CO₂, dry chemical, or foam for extinction.

STORAGE & DISPOSAL

Store in a well-ventilated place. Keep container tightly closed. Store locked up. Dispose of contents/container to an approved waste disposal plant.

NOTICE: Read Safety Data Sheet for additional information.

EMERGENCY CONTACT: Chemtrec 1-800-424-9300



CANCER
DIAGNOSTICS INC.

4300 Emperor Blvd - 400 Durham, NC 27703

Tel: 1-877-846-5393 Fax: 1-877-817-1716

www.cancerdiagnostics.com

Made in USA