



Stealth™ Safety Scalpels

| Catalog# | Description |
|----------|---|
| #SL0022 | #22 Disposable, Retractable Safety Scalpels, CS/100 |
| #SL0060 | #60 Disposable, Retractable Safety Scalpels, CS/100 |
| #SL0070 | #70 Disposable, Retractable Safety Scalpels, CS/100 |



Intended Use

Stealth™ Safety Scalpels feature an integrated rigid plastic handle with locking protective sheath. The sheath slides down and locks in place to expose the sterile scalpel blade. Slide the sheath back up for protection while not in use, or for disposal. The sheath will again, lock in place. The handle also features a metric 6-7cm ruler. Dispose when dull.

SAFETY GUARD SCALPELS

How to use Safety Scalpel

- 1 OPEN**
Press button.
Do not push forward.
- 2 READY FOR USE**
Pull guard forward to expose the blade.
Listen for click when the blade is fully opened.
Now it is ready for use.
- 3 CLOSE**
Push guard forward to cover the blade.
Listen for click.
- 4 SAFETY LOCK**
Hold one or two sides of guard.
Push forward until click for safety lock.
- 5 DISPOSE**
Dispose the scalpel in a suitable container.



STERILE R



Sterility guaranteed unless package has been damaged or seal is broken.