



Dermosin™

Skin Eosin

REF #ST0008 1 Gallon (3.8L)

LOT



For In Vitro Diagnostics Use



Date Received: _____ Date Opened: _____

Dermosin™ is an alcoholic skin eosin designed for use in Dermatopathology laboratories. Dermosin™ contains the appropriate amount of acid for proper counterstaining of epidermis and dermis and produces brilliant counterstaining with excellent differentiation of cell cytoplasm and other non-nuclear tissue components. The epidermis shows crisp nuclear staining, and parakeratotic and hyperkeratotic squamous cells demonstrate the proper blend of basophilia and eosinophilia. Erythrocytes are red; smooth muscle is dull pink; connective tissues are orange-pink; and cytoplasm of cells varies from pink to a "metachromatic" purplish pink. Dermosin™ stains the non-nuclear connective tissues with brilliance.



DANGER

HAZARD STATEMENTS

Causes skin irritation. Causes serious eye irritation. Causes damage to organs (respiratory system). May cause damage to organs (respiratory system, central nervous system, liver, blood) through prolonged or repeated exposure. Highly flammable liquid and vapor.

PRECAUTIONARY STATEMENTS - PREVENTION

Wash body thoroughly after handling. Wear protective gloves. Wear eye and face protection. Do not breathe dust, vapors. Do not eat, drink, or smoke when using this product. Keep away from heat sources and open flame. No smoking. Keep container tightly closed. Ground/bond container and receiving equipment. Use explosion-proof electrical, ventilating, lighting and equipment. Use only non-sparking tools. Take precautionary measures against static discharge.

PRECAUTIONARY STATEMENTS - RESPONSE

IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue Rinsing. If eye irritation persists: get medical attention. If exposed or concerned: call a doctor. Call a doctor if you feel unwell.

IF ON SKIN: Wash with plenty of water. Specific treatment, see first aid section on SDS. If skin irritation or rash occurs: get medical attention. Take off all contaminated clothing and wash it before reuse.

IN CASE OF FIRE: Use CO₂, dry chemical, or foam for extinction.

STORAGE & DISPOSAL

Store locked up at room temperature in a well ventilated place. Keep container tightly closed. Dispose of contents/container to an approved waste disposal plant.

NOTICE: Read Safety Data Sheet for additional information.

EMERGENCY CONTACT: Chemtrec 1-800-424-9300



4300 Emperor Blvd - 400 Durham, NC 27703

Tel: 1-877-846-5393 Fax: 1-877-817-1716

www.cancerdiagnostics.com

Made in USA

MX1339 Dermosin 1 Gal AW-1039 Rev.B 2020-07-02