



3-Stain Dip Kit SOLUTION A

Pre-Filled 4-Slide Jar



DANGER. Highly flammable liquid and vapor. Causes damage to organs(respiratory system). May causes organ (liver) damage through repeated or prolonged exposure. Causes skin and serious eye irritation. Read SDS for further hazard and safety information. Per OSHA store in product packaging until use.

Begin here, by placing air-dried slides

7-10 Seconds

Proceed to Red, Solution B

1



3-Stain Dip Kit SOLUTION A

Pre-Filled 4-Slide Jars

3STK-ABC



DANGER HAZARD STATEMENTS: Highly flammable liquid and vapor. Causes damage to organs(respiratory system). May causes organ (liver) damage through repeated or prolonged exposure. Causes skin and serious eye irritation. **PRECAUTIONARY STATEMENTS - PREVENTION:** Wash body thoroughly after handling. Wear protective gloves. Wear eye and face protection. Do not breathe dust, vapors. Do not eat, drink, or smoke when using this product. Keep away from heat sources and open flame. No smoking. Keep container tightly closed. Ground/bond container and receiving equipment. Use explosion-proof electrical, ventilating, lighting and equipment. Use only non-sparking tools. Take precautionary measures against static discharge. **RESPONSE:** Call a poison center/doctor if you feel unwell. If exposed or concerned: Call a poison center/doctor. **IF IN EYES:** Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. if eye irritation persists: Get medical advice/attention. Call a doctor if exposed or concerned, or if you feel unwell. **IF ON SKIN:** Wash with plenty of water. Specific treatment, see first aid section on SDS. If skin irritation or rash occurs: Get medical attention. Take off immediately all contaminated clothing and wash it before reuse. **IN CASE OF FIRE:** Use CO₂, dry chemical or foam for extinction. **STORAGE & DISPOSAL:** Store in a well-ventilated place. Keep cool. Store locked up. Dispose of contents/container in accordance with local/regional/national/international regulations. **NOTICE:** Read Safety Data Sheet for additional information.

EMERGENCY CONTACT: Chemtrec 1-800-424-9300

For In Vitro Diagnostics Use

PER OSHA: STORE PRODUCT IN HOLDING TRAY. DO NOT DISCARD



4300 Emperor Blvd - 400 Durham, NC 27703

Tel: 1-877-846-5393 Fax: 1-877-817-1716

cancerdiagnostics.com Made in USA

2



3-Stain Dip Kit SOLUTION B

Pre-Filled 4-Slide Jars



NON HAZARDOUS CHEMICAL: IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses. IF ON SKIN: Rinse skin with water/shower. IF INHALED: Remove to fresh air. keep victim in position comfortable for breathing. Per OSHA store in product packaging until use.

Cancer Diagnostics Inc. 4300 Emperor Blvd. #400 Durham, NC,
27703 Tel: (877) 846-5393

cancerdiagnostics.com

Place Slides in solution using gentle agitation

7-10 Seconds

Proceed to Blue, Solution C

2



3-Stain Dip Kit SOLUTION B

Pre-Filled 4-Slide Jars

3STK-ABC



NON HAZARDOUS CHEMICAL - FIRST AID:

IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue Rinsing. If eye irritation persists: get medical advice/attention. **IF ON SKIN:** Remove/Take off all contaminated clothing and wash it before reuse. Rinse skin with water/shower. If skin irritation occurs: get medical advice/attention. Call a POISON CENTER or doctor if you feel unwell. **IF INHALED:** Remove victim to fresh air and keep at rest in a position comfortable for breathing. **IN CASE OF FIRE:** Use CO₂, dry chemical, or foam for extinction. **STORAGE & DISPOSAL** Store locked up in a well ventilated place. Keep container tightly closed. Dispose of contents/container to an approved waste disposal plant.

NOTICE: Read Safety Data Sheet for additional information.

EMERGENCY CONTACT: Chemtrec 1-800-424-9300

For In Vitro Diagnostics Use

PER OSHA: STORE PRODUCT IN HOLDING TRAY. DO NOT DISCARD



4300 Emperor Blvd - 400 Durham, NC 27703

Tel: 1-877-846-5393 Fax: 1-877-817-1716

cancerdiagnostics.com Made in USA

3



3-Stain Dip Kit SOLUTION C

Pre-Filled 4-Slide Jars



NON HAZARDOUS CHEMICAL: IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses. IF ON SKIN: Rinse skin with water/shower. IF INHALED: Remove to fresh air. keep victim in position comfortable for breathing. Per OSHA store in product packaging until use.

Cancer Diagnostics Inc. 4300 Emperor Blvd. #400 Durham, NC,
27703 Tel: (877) 846-5393

cancerdiagnostics.com

Place Slides in solution using gentle agitation

7-10 Seconds

Wash gently in running tap water

1 Second

Allow to air dry, stand Semi-Vertical

2-5 Minutes

3



3-Stain Dip Kit SOLUTION C

Pre-Filled 4-Slide Jars

3STK-ABC



NON HAZARDOUS CHEMICAL - FIRST AID:

IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue Rinsing. If eye irritation persists: get medical advice/attention. **IF ON SKIN:** Remove/Take off all contaminated clothing and wash it before reuse. Rinse skin with water/shower. If skin irritation occurs: get medical advice/attention. Call a POISON CENTER or doctor if you feel unwell. **IF INHALED:** Remove victim to fresh air and keep at rest in a position comfortable for breathing. **IN CASE OF FIRE:** Use CO₂, dry chemical, or foam for extinction. **STORAGE & DISPOSAL** Store locked up in a well ventilated place. Keep container tightly closed. Dispose of contents/container to an approved waste disposal plant.

NOTICE: Read Safety Data Sheet for additional information.

EMERGENCY CONTACT: Chemtrec 1-800-424-9300

For In Vitro Diagnostics Use

PER OSHA: STORE PRODUCT IN HOLDING TRAY. DO NOT DISCARD



4300 Emperor Blvd - 400 Durham, NC 27703

Tel: 1-877-846-5393 Fax: 1-877-817-1716

cancerdiagnostics.com Made in USA

