



Saccomanno's Fixative

CYTOLOGY FIXATIVE

- #SF0038 1 GAL (3.8L)
- #SF0038-WP 1 GAL (3.8L) w/ PUMP
- #SF0037 1 GAL (3.8L), 4/CS
- #SF0037-WP 1 GAL (3.8L), 4/CS w/ PUMP
- #SF0537 5 GAL (19L) CUBE

For In Vitro Diagnostics Use

Date Received: _____ Date Opened: _____

Saccomanno's Fixative (modified) is prepared particularly for collection and fixation of sputum specimens. Our modification of the fixative combines the coagulant fixative qualities of alcohol with other fixation enhancements to produce better fixation of nucleoplasm, nucleoli and cytoplasm. Erythrocytes are lysed and do not obscure the nucleated cells of interest. Traditional cytology stains produce distinct nuclear and cytoplasmic membranes, and nucleoli are distinct and crisp. This is the fixative of choice for collection and fixation of sputum specimens.



DANGER HAZARD STATEMENTS

Causes skin and serious eye irritation. Causes damage to organs (respiratory system). May cause damage to organs (respiratory system, central nervous system, liver, blood) through prolonged or repeated exposure. Flammable liquid and vapor.

PRECAUTIONARY STATEMENTS - PREVENTION

Wash body thoroughly after handling. Wear protective gloves. Wear eye and face protection. Do not breathe dust, vapors. Do not eat, drink or smoke when using this product. Keep away from heat sources and open flame. No smoking. Keep container tightly closed. Ground/bond container and receiving equipment. Use explosion-proof electrical, ventilating, lighting and equipment. Use only non-sparking tools. Take precautionary measures against static discharge.

PRECAUTIONARY STATEMENTS - RESPONSE

IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue Rinsing. If eye irritation persists: get medical attention. If exposed or concerned: Call a doctor. Call a doctor if you feel unwell.

IF ON SKIN: Wash with plenty of water. Specific treatment, see first aid section on SDS. If skin irritation or rash occurs: get medical attention. Take off immediately all contaminated clothing and wash it before reuse.

IN CASE OF FIRE: Use CO₂, dry chemical, or foam for extinction.

STORAGE & DISPOSAL

Store locked up. Store in a well-ventilated place. Keep cool. Dispose of contents/container to an approved waste disposal plant.

NOTICE: Read Safety Data Sheet for additional information.

EMERGENCY CONTACT: Chemtrec 1-800-424-9300



4300 Emperor Blvd - 400 Durham, NC 27703

Tel: 1-877-846-5393 Fax: 1-877-817-1716

www.cancerdiagnostics.com

Made in USA