



CANCER
DIAGNOSTICS, INC.

 PUREVIEW[®]

Hematoxylin and Eosin
Staining System

 PH Blue[™]

Concentrate

REF

16702-C 500mL each


16702-C4 500mL, 4/case



Order Toll Free: (877) 846 5393

Fax: (877) 817 1716

www.cancerdiagnostics.com

 Cancer Diagnostics, Inc.
4300 Emperor Blvd-400.
Durham, NC 27703

NET 500 ML

Made in USA

DIRECTIONS:

PH Blue is a concentrated reagent and should be diluted before use. To obtain a working solution, pour two measures of concentrate into 500mL of distilled water (30mL). One measure equals fluid filled to first line in bottle cap. A small margin of error produced from using the cap for measurement is insignificant to the performance, pH, and strength of the reagent.

For *in vitro* diagnostic use only.

Store at room temperature (20°-30°C).

NON HAZARDOUS CHEMICAL

FIRST AID: If exposure by any route causes irritation get medical advice/attention.

IF IN EYES: Flush eyes with water as a precaution.

IF ON SKIN (OR HAIR): Wash off with plenty of water. Remove contaminated clothing and launder before reuse as a precaution.

IF INHALED: Move person to fresh air; give artificial respiration if breathing has stopped.

IF INGESTED: Never give anything by mouth to an unconscious person. Rinse mouth with water. Get medical attention if discomfort occurs.

IN CASE OF FIRE: Use water, CO₂ or foam to extinguish.

STORAGE: Store in cool, well-ventilated place. Keep lid tightly closed.

DISPOSAL: Dispose of contents/container in accordance with local regulations.

NOTICE: Read Safety Data Sheet for additional information.

24 Hour Emergency Contact:
CHEMTREC 800-424-9300
USA and Canada



Date Received: _____

Date Opened: _____