



# Neutral Buffered Formalin Concentrate (4:1)

## FIXATIVE

#FX1014 | GAL (3.8L), 4/CS

**D.O.T. Proper Shipping Name:**  
**UN2209 Formaldehyde Solutions, 8, III**  
**For In Vitro Diagnostics Use**

Date Received: \_\_\_\_\_ Date Opened: \_\_\_\_\_

Directions: Mix with water to product 10% Neutral Buffered Formalin. One gallon of concentrate is mixed with 4 gallons of water, to yield five gallons of 10% Neutral Buffered Formalin.



### **DANGER**

#### **HAZARD STATEMENTS**

Harmful if swallowed. Toxic if inhaled. Harmful in contact with skin. Causes severe skin burns and serious eye damage. May cause allergy or asthma symptoms or breathing difficulties if inhaled. May cause an allergic skin reaction. May cause genetic defects. May cause cancer. May cause damage to organs (nose, lungs) through single, prolonged, or repeated exposure.

#### **PRECAUTIONARY STATEMENTS - PREVENTION**

Wash body thoroughly after handling. Do not eat, drink, or smoke when using this product. Use only outdoors or in a well-ventilated area. Do not breathe dusts, mists, or vapors. Wear protective gloves and clothing. Wear eye and face protection. Wear NIOSH approved respiratory protection. Contaminated work clothing must not be allowed out of the workplace. Obtain special instructions before use. Do not handle until all safety precautions have been read and understood.

#### **PRECAUTIONARY STATEMENTS - RESPONSE**

**IF IN EYES:** Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue Rinsing.

**IF ON SKIN:** Wash with plenty of water. Call a doctor if you feel unwell. Take off immediately all contaminated clothing and wash it before reuse. Immediately call a doctor. If skin irritation or rash occurs: get medical attention. If exposed or concerned: get medical attention/call a doctor.

**IF INHALED:** If breathing is difficult, remove person to fresh air and keep comfortable for breathing. If experiencing respiratory symptoms: call a doctor. Call a doctor. Specific treatment, see first aid section on SDS.

**IF SWALLOWED:** Call a doctor if you feel unwell. Rinse mouth. Do NOT induce vomiting.

#### **STORAGE & DISPOSAL**

Store in a well-ventilated place. Keep container tightly closed. Store locked up. Dispose of contents/container to an approved waste disposal plant.

**NOTICE:** Read Safety Data Sheet for additional information.

**EMERGENCY CONTACT:** Chemtrec 1-800-424-9300

 **CANCER**  
DIAGNOSTICS INC.  
4300 Emperor Blvd - 400 Durham, NC 27703  
Tel: 1-877-846-5393 Fax: 1-877-817-1716

[www.cancerdiagnostics.com](http://www.cancerdiagnostics.com)

**Made in USA**