



# Bouin's Solution

## FIXATIVE

#FX1035 1 GAL (3.8L)

#FX1036 5 GAL (19L) CUBE

For In Vitro Diagnostics Use

Date Received: \_\_\_\_\_ Date Opened: \_\_\_\_\_

Bouin's is intended as a special fixative or a mordant for tissues to be examined by light microscopy. It is useful for surgical tissues and necropsy specimens. Trichrome stains are enhanced by Bouin's. Tissue sections show improved staining of non-nuclear connective tissue and smooth muscle elements following mordant treatment with Bouin's.



### DANGER

#### HAZARD STATEMENTS

Toxic if inhaled. Harmful if swallowed. Causes serious eye damage. May cause allergy or asthma symptoms or breathing difficulties if inhaled. May cause an allergic skin reaction. May cause genetic defects. May cause cancer. May cause damage to organs (lungs, nose). May cause damage to organs (lungs, nose) through prolonged or repeated exposure.

#### PRECAUTIONARY STATEMENTS - PREVENTION

Do not breathe dust, vapors. Use only outdoors or in a well-ventilated area. Wash body thoroughly after handling. Do not eat, drink, or smoke when using this product. Wear eye protection, face protection. Wear NIOSH approved respiratory protection. Contaminated work clothing must not be allowed out of the workplace. Wear protective gloves. Obtain special instructions before use. Do not handle until all safety precautions have been read and understood. Wear protective gloves and clothing. Wear eye and face protection.

#### PRECAUTIONARY STATEMENTS - RESPONSE

**IF IN EYES:** Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue Rinsing Immediately call a doctor.

**IF ON SKIN:** Wash with plenty of water. If skin irritation or rash occurs: Get medical attention. Take off all contaminated clothing and wash it before reuse. If exposed or concerned: Get medical attention. If exposed or concerned: Call a doctor. Call a doctor if you feel unwell.

**IF INHALED:** If breathing is difficult, remove person to fresh air and keep comfortable for breathing. If experiencing respiratory symptoms: Call a doctor. Call a doctor. Specific treatment, see first aid section on SDS.

**IF SWALLOWED:** Call a doctor if you feel unwell. Rinse mouth.

#### STORAGE & DISPOSAL

Store in a well-ventilated place. Keep container tightly closed. Store locked up. Dispose of contents/container to an approved waste disposal plant.

**NOTICE:** Read Safety Data Sheet for additional information.

**EMERGENCY CONTACT:** Chemtrec 1-800-424-9300



CANCER  
DIAGNOSTICS INC

4300 Emperor Blvd - 400 Durham, NC 27703

Tel: 1-877-846-5393 Fax: 1-877-817-1716

www.cancerdiagnostics.com

Made in USA