



# CoverSeal™-T

## TOLUENE MOUNTING MEDIUM

#FX2178 500ML

For In Vitro Diagnostics Use

Date Received: \_\_\_\_\_ Date Opened: \_\_\_\_\_

CoverSeal™-T is a toluene based mounting medium designed to be used with staining procedures using Xylene or ZeroXylene™ as clearing agents. It is miscible with all routine clearing reagents and has an optimum refractive index.



### DANGER

#### HAZARD STATEMENTS

Harmful if swallowed. Causes skin and serious eye irritation. Suspected of damaging fertility of the unborn child. May cause damage to organs (central nervous system). May cause damage to organs (liver) through prolonged or repeated exposure. Highly flammable liquid and vapor.

#### PRECAUTIONARY STATEMENTS - PREVENTION

Wash body thoroughly after handling. Do not eat, drink, or smoke when using this product. Obtain special instructions before use. Do not handle until all safety precautions have been read and understood. Wear protective gloves and clothing. Wear eye and face protection. Do not breathe dust, vapors. Keep away from heat sources and open flame. No smoking. Keep container tightly closed. Ground/bond container and receiving equipment. Use explosion-proof electrical, ventilating, lighting and equipment. Use only non-sparking tools. Take precautionary measures against static discharge.

#### PRECAUTIONARY STATEMENTS - RESPONSE

**IF IN EYES:** Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue Rinsing. If eye irritation persists: get medical attention. If exposed or concerned: get medical attention/call a doctor. Call a doctor if you feel unwell.

**IF ON SKIN:** Wash with plenty of water. Specific treatment, see first aid section on SDS. If skin irritation or rash occurs: get medical attention. Take off all contaminated clothing and wash it before reuse.

**IF SWALLOWED:** Call a doctor if you feel unwell. Rinse mouth.

**IN CASE OF FIRE:** Use CO<sub>2</sub>, dry chemical, or foam for extinction.

#### STORAGE & DISPOSAL

Store locked up. Store in a well-ventilated place. Keep cool. Dispose of contents/container to an approved waste disposal plant.

**NOTICE:** Read Safety Data Sheet for additional information.

**EMERGENCY CONTACT:** Chemtrec 1-800-424-9300



CANCER  
DIAGNOSTICS INC.

4300 Emperor Blvd - 400 Durham, NC 27703

Tel: 1-877-846-5393 Fax: 1-877-817-1716

www.cancerdiagnostics.com **Made in USA**