

Shake well before using

Intended use: For defining margins of excised tissue.

Apply dyes to fresh or formalin fixed tissue. Only small amounts of dye are necessary for effective marking. Do not apply to living patients. Refer to SDS for more information.

24 Hour Emergency Contact

CHEMTEC USA and Canada 800-424-9300

59°F (15°C)  86°F (30°C)

Warning: Causes skin irritation. Causes eye irritation. Wash skin thoroughly after handling. Wear eye protection/face protection. Take off contaminated clothing and wash it before reuse. **IF IN EYES:** Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. **IF ON SKIN:** Wash with plenty of soap and water. If eye or skin irritation persists: Get medical advice/attention. Store locked up. Dispose of contents/container in accordance with local regulations.

 Cancer Diagnostics, Inc. 4300 Emperor Blvd - 400 Durham, NC 27703 USA

 mit Europa GmbH Langerhagener Str. 7
D-30855 Langerhagen Germany +49-511-3908 9530



  MADE IN USA
(877.646.5393)

LOT



H01213 0.5oz Flip-Top, Yellow A99-15582 Rev.C 2020-02-05



CANCER
DIAGNOSTICS INC.

CDI's[®]
Tissue Marking
Dyes

YELLOW

REF 0723-5

0.5 OZ. (15ML) FLIP-TOP

cancerdiagnostics.com