



# SS1038-VO

## Picro-Sirius Red Stain Kit (For Collagen)

**Description:** The Picro-Sirius Red Stain Kit (For Collagen) is intended for use in the histological visualization of collagen I and III fibers in addition to muscle in tissue sections. The PSR stain may be viewed using standard light microscopy or polarized light resulting in birefringence of the collagen fibers to distinguish between type I and type III.


**Light Microscopy**

Collagen: Red  
Muscle Fibers: Yellow  
Cytoplasm: Yellow

**Polarized Light Microscopy**

Type I (Thick fibers) Yellow-Orange Birefringence  
Type III (Thin fibers) Green Birefringence

**Uses/Limitations:** Not to be taken internally.  
For In-Vitro Diagnostic use only.  
Histological applications.  
Do not use if reagents become cloudy.  
Do not use past expiration date.  
Use caution when handling reagents.  
Non-Sterile.

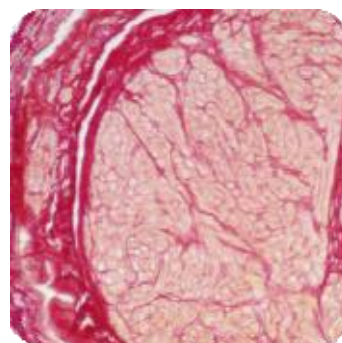

**Control Tissue:** Lung  
Uterus  
Muscle  
Kidney

**Availability/Contents:**

<u>Kit Contents</u>	<u>Volume</u>	<u>Storage</u>
Picro-Sirius Red Solution	250 ml	18-25°C
Acetic Acid Solution (0.5%)	250 ml x 2	18-25°C

Ordering information regarding individual components on back page!

**Precautions:** Avoid contact with skin and eyes.  
Harmful if swallowed.  
Follow all Federal, State, and local regulations regarding disposal.  
Use in chemical fume hood whenever possible.

Storage: 18° C  25° C



## SS1038-VO


### Procedure:

1. Deparaffinize sections if necessary and hydrate to distilled water.
2. Apply adequate Picro-Sirius Red Solution to completely cover tissue section and incubate for 60 minutes.
3. Rinse slide quickly in two changes of Acetic Acid Solution (0.5%).
4. Rinse slide using absolute alcohol.
5. Dehydrate in 2 changes of absolute alcohol, clear, and mount in synthetic resin.

### References:

1. Puchtler H., Waldrop F.S., Valentine L.S. Polarization microscopic studies of connective tissue stained with picro-sirius red FBA. Beitr Path. 1973; 150, pages 174-187.
2. Junqueira L.C.U., Bignolas G., Brentani R.R. Picrosirius staining plus polarization microscopy, a specific method for collagen detection in tissue sections. Histochemistry J. 1979, 11, pages 447-455.
3. Whittaker P. Polarized light microscopy in biomedical research. Microscopy and Analysis 1995; 44, pages 15-17.

Description:	Volume
Picro-Sirius Red Solution	250 ml
	500 ml
	1000 ml
Acetic Acid Solution	250 ml
	500 ml
	1000 ml

Storage: 18° C  25° C

