



Prostate Biopsy Collection Kit

10% Neutral Buffered Formalin

#PP1016 20mL, 16 Bottle/Kit

Preparing Prostate Specimens for Transport

1. Complete requisition form, only one patient per requisition form.
2. Label each vial with the appropriate information. Use blue or black ink only.
3. Place each specimen into the corresponding vial. Secure cap tightly and place each vial into the appropriate section of the specimen tray. Multiple core specimens from the same area should be placed in the same collection vial. Be sure to discard any unused formalin collection vials to avoid confusion.
4. Enlose the Test Requisition and secure Prostate Kit for transport.

For In Vitro Diagnostics Use

Date Received: _____ Date Opened: _____



DANGER **CONTAINS FORMALDEHYDE** **HAZARD STATEMENTS**

Harmful if inhaled. Causes skin irritation. Causes serious eye damage. May cause allergy or asthma symptoms or breathing difficulties if inhaled. May cause an allergic skin reaction. May cause genetic defects. May cause cancer. May cause damage to organs (lungs, nose) through prolonged or repeated exposure.

PRECAUTIONARY STATEMENTS - PREVENTION

Do not breathe dust, vapors. Use only outdoors or in a well-ventilated area. Wear NIOSH approved respiratory protection. Wash body thoroughly after handling. Wear protective gloves and clothing. Wear eye and face protection. Contaminated work clothing must not be allowed out of the workplace. Obtain special instructions before use. Do not handle until all safety precautions have been read and understood. Do not eat, drink, or smoke when using this product.

PRECAUTIONARY STATEMENTS - RESPONSE

IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing, immediately call a doctor.

IF ON SKIN: Wash with plenty of water. Specific treatment, see First Aid Measures on SDS. If skin irritation or rash occurs: get medical attention. Take off all contaminated clothing and wash it before reuse.

IF INHALED: If breathing is difficult, remove person to fresh air and keep comfortable for breathing. If experiencing respiratory symptoms: call a doctor. If exposed or concerned: get medical attention and/or call a doctor if you feel unwell.

STORAGE & DISPOSAL

Store locked up. Dispose of contents/container to an approved waste disposal plant.

NOTICE: Read Safety Data Sheet for additional information.

EMERGENCY CONTACT: Chemtrec 1-800-424-9300



CANCER
DIAGNOSTICS INC.

4300 Emperor Blvd - 400 Durham, NC 27703

Tel: 1-877-846-5393 Fax: 1-877-817-1716

www.cancerdiagnostics.com

Made in USA