



Prostate-Pak™

Biopsy Collection Kit

10% Neutral Buffered Formalin

REF #PP1010 10 Jar Kit

LOT



For In Vitro Diagnostic Use

59°F (15°C) — 86°F (30°C)

Date Received: _____ Date Opened: _____

Preparing Prostate Specimens for Transport

1. Complete requisition form, only one patient per requisition form.
2. Label each vial with the appropriate information. Use blue or black ink only.
3. Place each specimen into the corresponding vial. Secure cap tightly and place each vial into the appropriate section of the specimen tray. Multiple core specimens from the same area should be placed in the same collection vial. Be sure to discard any unused formalin collection vials to avoid confusion.
4. Enlose the Test Requisition and secure Prostate Kit for transport.



DANGER

HAZARD STATEMENTS - CONTAINS FORMALDEHYDE

Harmful if inhaled. Causes skin irritation. Causes serious eye damage. May cause an allergic skin reaction. May cause cancer. Causes damage to organs. Combustible liquid.

PRECAUTIONARY STATEMENTS - PREVENTION

Obtain special instructions before use. Do not handle until all safety precautions have been read and understood. Use personal protective equipment as required. Use only outdoors or in a well-ventilated area. Wash face, hands and any exposed skin thoroughly after handling. Contaminated work clothing should not be allowed out of the workplace. Wear protective gloves. Do not breathe dust/fume/gas/mist/vapors or spray. Do not eat, drink or smoke when using this product. Keep away from heat/sparks/open flames/hot surfaces. — No smoking.

PRECAUTIONARY STATEMENTS - RESPONSE

IF exposed: Call a POISON CENTER or doctor/physician.

IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Immediately call a POISON CENTER or doctor / physician.

IF ON SKIN: Wash with plenty of soap and water. Take off contaminated clothing and wash before reuse. If skin irritation or rash occurs: Get medical advice/attention.

IF INHALED: Remove victim to fresh air and keep at rest in a position comfortable for breathing. Call a POISON CENTER or doctor/physician if you feel unwell.

IN CASE OF FIRE: Use CO₂, dry chemical, or foam for extinction.

STORAGE & DISPOSAL

Store locked up in a well ventilated place at room temperature. Keep cool. Dispose of contents/container to an approved waste disposal plant.

NOTICE: Read Safety Data Sheet for additional information.

EMERGENCY CONTACT: Chemtrec 1-800-424-9300



116 Page Point Circle Durham, NC 27703

Tel: 1-877-846-5393 Fax: 1-877-817-1716

www.cancerdiagnostics.com

Made in USA

MX2164 Prostate Pack 10 Jar AWW-3059479 Rev.B 2024-02-23