



Fast Fix™

FIXATIVE

- #FX1057 1000ML
- #FX1053 1 GAL (3.8L)
- #FX1054 1 GAL (3.8L), 4/CS
- #FX1055 2.5 GAL (9.5L) CUBE
- #FX1056 5 GAL (19L) CUBE

For In Vitro Diagnostics Use

Date Received: _____ Date Opened: _____

Fast Fix™ is a rapid fixative designed specifically to be used at the grossing (cutting) station. It penetrates and fixes fatty tissues rapidly. It is compatible with all other primary fixatives and with NBF. This is the fixative to use at the grossing station to improve fixation, dehydration, clearing and wax infiltration. It works ideally when used in combination with PureView® Series Processing 1™ and 2™ on the processor.



DANGER CONTAINS FORMALDEHYDE HAZARD STATEMENTS

Toxic if inhaled. Harmful if swallowed. Causes severe skin burns and serious eye damage. May cause allergy or asthma symptoms or breathing difficulties if inhaled. May cause an allergic skin reaction. May cause genetic defects. May cause cancer. May cause damage to organs (nose, lungs) through single, prolonged, or repeated exposure.

PRECAUTIONARY STATEMENTS - PREVENTION

Use only outdoors or in a well-ventilated area. Wash body thoroughly after handling. Do not eat, drink, or smoke when using this product. Do not breathe dusts, mists, or vapors. Wear protective gloves and clothing. Wear eye and face protection. Wear eye protection, face protection. Wear NIOSH approved respiratory protection. Contaminated work clothing must not be allowed out of the workplace. Obtain special instructions before use. Do not handle until all safety precautions have been read and understood. Do not breathe dust, vapors.

PRECAUTIONARY STATEMENTS - RESPONSE

IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue Rinsing.

IF ON SKIN: Rinse skin with water/shower. Take off immediately all contaminated clothing and wash it before reuse. Immediately call a doctor. If skin irritation or rash occurs: get medical attention. If exposed or concerned: get medical attention/call a doctor. Call a doctor if you feel unwell.

IF INHALED: If breathing is difficult, remove person to fresh air and keep comfortable for breathing. If experiencing respiratory symptoms: call a doctor. Call a doctor. Specific treatment, see first aid section on SDS.

IF SWALLOWED: Call a doctor if you feel unwell. Rinse mouth. Do NOT induce vomiting.

STORAGE & DISPOSAL

Store in a well-ventilated place. Keep container tightly closed. Store locked up. Dispose of contents/container to an approved waste disposal plant.

NOTICE: Read Safety Data Sheet for additional information.

EMERGENCY CONTACT: Chemtrec 1-800-424-9300

4300 Emperor Blvd - 400 Durham, NC 27703

Tel: 1-877-846-5393 Fax: 1-877-817-1716

www.cancerdiagnostics.com

Made in USA

