



Davidson Fixative

REF #FX1041 1 Gallon (3.8L) CS/4

LOT



For In Vitro Diagnostic Use



Date Received: _____ Date Opened: _____

Davidson's Fixative is a rapid fixative which gives good nuclear detail with minimal formalin pigment. Specimens cut and stain as if fixed in ordinary formalin. Davidson's Fixative contains no mercury or other metals. Davidson's Fixative is particularly useful for preparing tumors, bone marrow specimens, gynecologic material, fatty breast, and medical biopsies. It may be used for testicular biopsies and for overnight fixation of fat containing lymph nodes. Fixation of small specimens is rapid. Exposure to the fixative should be limited to 24 hours. Tissue may be transferred to neutral, buffered formalin or 70% alcohol for storage.



DANGER HAZARD STATEMENTS

Harmful if swallowed. Harmful if inhaled. Causes severe skin burns and eye damage. May cause an allergic skin reaction. May cause cancer. Causes damage to organs. Flammable liquid and vapor.

PRECAUTIONARY STATEMENTS - PREVENTION

Obtain special instructions before use. Do not handle until all safety precautions have been read and understood. Use personal protective equipment as required. Wash face, hands and any exposed skin thoroughly after handling. Do not eat, drink or smoke when using this product. Use only outdoors or in a well-ventilated area. Do not breathe dust/fume/gas/mist/vapors/spray. Contaminated work clothing should not be allowed out of the workplace. Wear protective gloves. Keep away from heat/sparks/open flames/hot surfaces. — No smoking. Keep container tightly closed. Ground/bond container and receiving equipment. Use explosion-proof electrical/ventilating/lighting/equipment. Use only non-sparking tools. Take precautionary measures against static discharge.

PRECAUTIONARY STATEMENTS - RESPONSE

Immediately call a POISON CENTER or doctor/physician.

IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue Rinsing.

IF ON SKIN: Remove/Take off immediately all contaminated clothing. Rinse skin with water/shower. If skin irritation or rash occurs: Get medical advice/attention. Wash contaminated clothing before reuse.

IF INHALED: Remove victim to fresh air and keep at rest in a position comfortable for breathing.

IF SWALLOWED: Call a POISON CENTER or doctor/physician if you feel unwell. Rinse mouth. Do NOT induce vomiting.

STORAGE & DISPOSAL

Store locked up in a well-ventilated place at room temperature. Keep cool. Dispose of contents/container to an approved waste disposal plant.

NOTICE: Read Safety Data Sheet for additional information.

EMERGENCY CONTACT: Chemtrec 1-800-424-9300



4300 Emperor Blvd - 400 Durham, NC 27703

Tel: 1-877-846-5393 Fax: 1-877-817-1716

www.cancerdiagnostics.com **Made in USA**

MX1982 Davidsons Fixative AW-4015605 Rev B 2022-03-22