CaviCide™

Step 1: Clean before disinfecting


1. Spray CaviCide directly onto surface to be cleaned.



2. Wipe with clean towel to pre-clean surface of all gross debris.



3. Dispose of towel in accordance with Federal, State, and local regulations for infectious materials disposal.

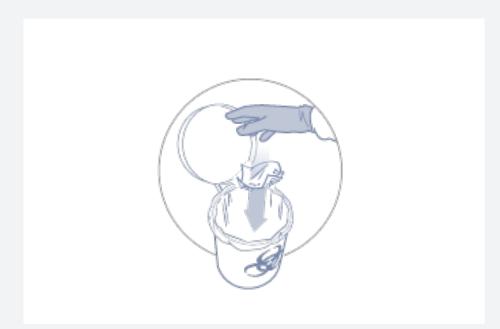

CaviCide is an intermediate level disinfectant recommended for use on hard, non-porous surfaces in healthcare settings.

- Effective against TB and MRSA in 3 minutes
- Effective against HIV-1, HBV, and HCV in 2 minutes
- For use on hard non-porous surfaces
- Cleaner and Disinfectant

Step 2: Disinfect after cleaning

1. Spray CaviCide directly onto surface, thoroughly wetting area. Leave surface wet for 3 minutes.



2. Wipe with clean towel. Once complete, dispose of used towelette in accordance with Federal, State, and local regulations for infectious materials disposal.



Ordering Information

Part No.	Description	Packaging
13-1002	CaviCide – 2 ounce spray	48 bottles / case
13-1008	CaviCide – 8 ounce spray	12 bottles / case
13-1024	CaviCide – 24 ounce spray	12 bottles / case
13-1000	CaviCide – 1 gallon	4 bottles / case
13-1025	CaviCide – 2.5 gallon	2 bottles / case
13-1055	CaviCide – 55 gallon	1 drum

