

**Diecut of Foil Inside**

**DISPOSABLE TISSUE MARKING DYE KIT**  
www.cancerdiagnostics.com



use and access (mini) kit on reverse side

**Outside**

**Color Key**



Color Key corresponds to dye locations on right

www.cancerdiagnostics.com

CDI's Tissue Dye Kit (mini) For orientation of excised surgical specimens

**Guidelines for Marking & Mapping Specimens**

Every user will find their own technique for marking tissue specimens. The user may use any color for any area of the specimen as long as the map is correct to maintain proper orientation.

We recommend a gross sheet such as this one, which can be downloaded in PDF format. Note the legend which uses numbers to be used in the mapping process.

Below are examples of marking and mapping tissue.

**Step 1**  
On the gross sheet, draw the specimen to be marked.

**Step 2**  
Gross the specimen in sections and mark the gross sheet accordingly.

**Step 3**  
Mark the specimen and note the map with the corresponding number in the legend.

**Continue**  
Continue to mark each edge of tissue and mark the map to correspond with the colors.

When all sides of each tissue have been marked and the map has corresponding numbers, the tissue is ready for processing.

This is only a representative technique that can be used, and any similar techniques will produce satisfactory results as long as the map corresponds to the marked tissue colors.

Color	Number
Black	1
Blue	2
Green	3
Red	4
Yellow	5
Orange	6
Violet	7

**Inside**



**USE THIS SIDE UP**

**Directions**

- Open at tab to expose foil
- Pierce foil to access dyes
- Use applicator of choice to apply dyes
- Detach gross board at perforation as needed
- Dispose when finished

**OPEN**

59°F (15°C) 86°F (30°C)

**Warning** Causes eye irritation. Causes skin irritation. Wash skin thoroughly after handling. Wear eye protection/ face protection. Take off contaminated clothing and wash it before reuse. Store locked up. **IF IN EYES:** Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. **IF ON SKIN:** Wash with plenty of soap and water. If eye or skin irritation persists: get medical advice/attention.



IVD

Cancer Diagnostics, Inc. 4300 Emperor Blvd - 400 Durham, NC 27703 USA EC REP mdi Europa GmbH Langenhagener Str. 71 D-30855 Langenhagen Germany +49-511-3908 9530

CDI's Tissue Dye Kit (mini) For orientation of excised surgical specimens

**Bio Board (mini)** (Detach as needed)

60 mm

50

40

30

20

10

5

1 2 3 4 5 6 7 8 9 10 cm

16

14

12

10

6


5

4

**BIOHAZARD**

(\* ONLY AFTER USE WITH BIOLOGIC SPECIMENS CONTAMINATED INFECTIOUS WASTE)

**Foil overlay Inside**

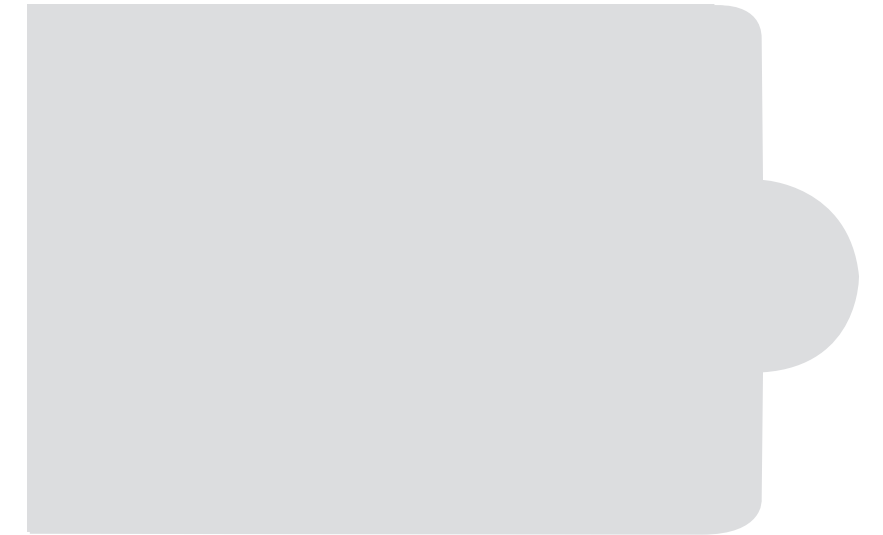

Crease

Perf

Crease

Perf

Crease

Perf