

Shake well before using

Apply dye to fresh or frozen fixed tissue. Only small amounts of dye are necessary for effective marking. Do not apply to living patients. Refer to SDS for more information.

TEAL

Warning: Causes skin irritation. Causes eye irritation. Wash skin thoroughly after handling. Wear eye and face protection. Take off contaminated clothing and wash it before reuse. Store Locked Up. Dispose of contents in accordance with local regulations.

IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. **IF ON SKIN:** Wash with plenty of soap and water. If red or skin irritation persists, Get medical advice or attention.



CANCER
DIAGNOSTICS, INC.

CDI's®

Tissue Marking Dye

APPLICATOR SERIES™



Cancer Diagnostics, Inc.
4200 Superior Blvd, #400, Durham NC 27705 USA



and Europa GmbH Langenloper Str. 71 D-30855
Langenhagen Germany +49 51 1 3908 9330



38F (15°C) 38F (15°C) 38F (15°C)



24 Hour Emergency Contact: USA and Canada 800-424-9300

MX1237 Applicator Series, Teal AW-17218 Rev.B 2020-02-05

REF 0726-9

.10 FL. OZ., (3ML), FINE LINE BRUSH CAP, PK/5

TEAL

cancerdiagnostics.com

MADE IN USA