

Apply dyes to fresh or formalin fixed tissue. Only small amounts of dye are necessary for effective marking. Do not apply to living patients. Refer to SDS for more information.


24 Hour Emergency Contact
CHEMTREC USA and Canada 800-424-9300

Kits

MD3000 7 Color kit 3 of each 7 colors (21 w/caps)
 MD2000 6 Color kit 3 of each 6 colors (18 w/caps)

Single Color: (PK/15)


- MD1001 Black
- MD1002 Blue
- MD1003 Green
- MD1004 Red
- MD1005 Yellow
- MD1006 Orange
- MD1007 Violet

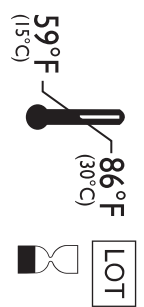

WARNING
 Causes skin irritation. Causes eye irritation **IF IN EYES**. Rinse with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. **IF ON SKIN**: Wash with plenty of soap and water. If eye or skin irritation persists: Get medical advice/attention. Store locked up. Store product in product packaging. Dispose of contents/container in accordance with local regulations.

MX1241 MD Squeezer Tray AW-MD3000-1373 Rev.B 2020-02-06

- ### Features
- Optimal dye consistency
 - Easy application
 - Packaging limits exposure to air
 - Bright dyes hold permanently through processing and staining

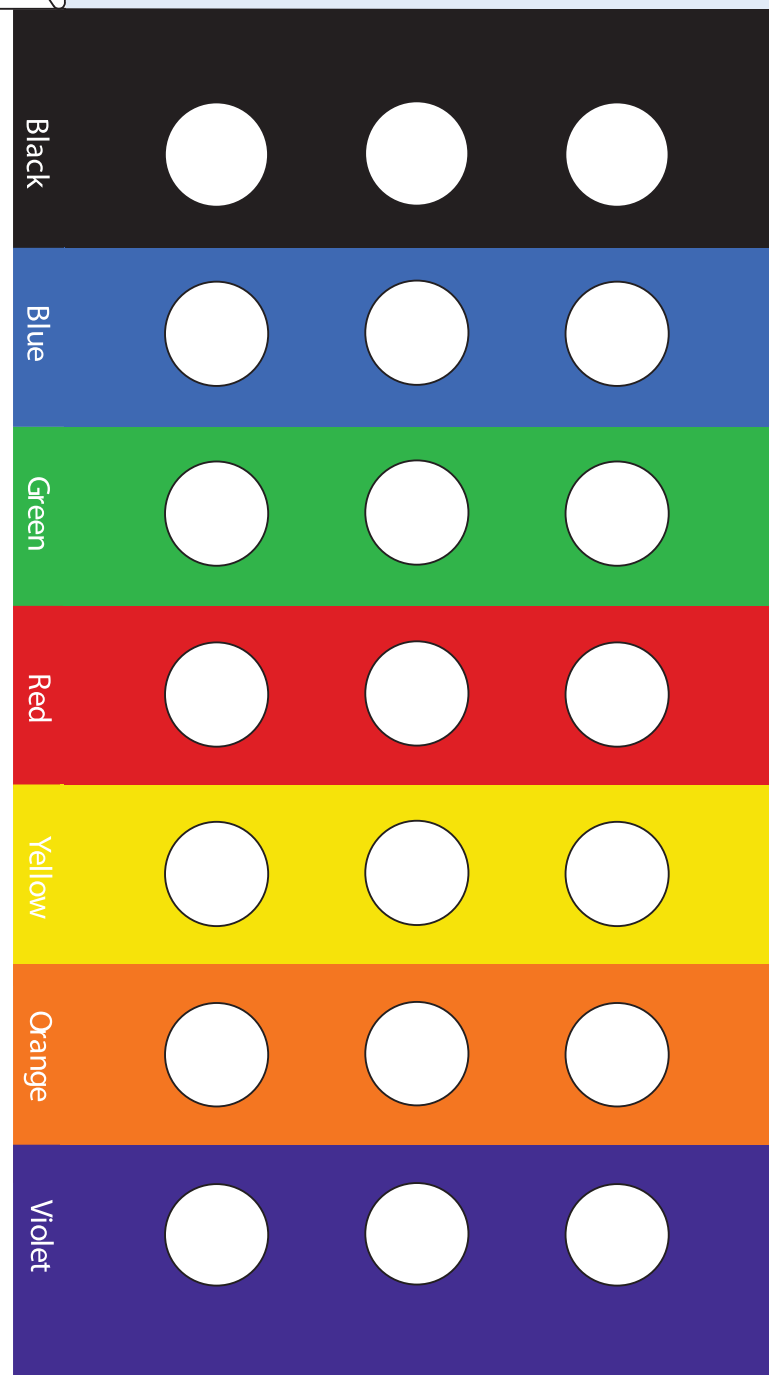

MADE IN USA
Order Toll Free: (877) 846 5393 www.cancerdiagnostics.com
 Cancer Diagnostics, Inc. 4300 Emperor Blvd - 400 Durham, NC 27703 USA
 mdi Europa GmbH Langenhagen Str. 71 D-30855 Langenhagen Germany +49-511-3908 9330
 EC REP



CDI's
 Tissue Marking Dyes®
Squeezers

- Pull off scored tip or snip.
- Dye can be squeezed into small receptacle and applied with applicator stick, or applied to large tissue directly.
- Recap unused portion.

