



# Acid Alcohol, 1%

REF #ST0017 | GAL (3.8L), CS/4

LOT



For In Vitro Diagnostics Use



Date Received: \_\_\_\_\_ Date Opened: \_\_\_\_\_

Formulated as a differentiator of Hematoxylin in regressive H&E staining procedures. This differentiator provides ideal differentiation of nuclear structures. Non-nuclear structures show clear staining by eosin with precise color variations by the eosin. Residual mucin staining by the hematoxylin is minimal to slight.



## **DANGER** **HAZARD STATEMENTS**

Causes skin irritation. Causes serious eye irritation. Causes damage to organs. Highly flammable liquid and vapor.

## **PRECAUTIONARY STATEMENTS - PREVENTION**

Wash face, hands and any exposed skin thoroughly after handling. Wear protective gloves/protective clothing/eye protection/face protection. Do not breathe dust/fume/gas/mist/vapors/spray. Do not eat, drink or smoke when using this product. Keep away from heat/sparks/open flames/hot surfaces. — No smoking. Keep container tightly closed. Ground/bond container and receiving equipment. Use explosion-proof electrical/ventilating/lighting/equipment. Use only non-sparking tools. Take precautionary measures against static discharge.

## **PRECAUTIONARY STATEMENTS - RESPONSE**

**IF IN EYES:** Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. If eye irritation persists: Get medical advice/attention.

**IF ON SKIN:** Remove/Take off immediately all contaminated clothing. Rinse skin with water/shower. If skin irritation occurs: Get medical advice/attention. Wash contaminated clothing before reuse.

IF exposed: Call a POISON CENTER or doctor/physician.

**IN CASE OF FIRE:** Use CO<sub>2</sub>, dry chemical, or foam for extinction.

## **STORAGE & DISPOSAL**

Store locked up, in a well ventilated place at room temperature. Keep cool.

Dispose of contents/container to an approved waste disposal plant.

**NOTICE:** Read Safety Data Sheet for additional information.

**EMERGENCY CONTACT:** Chemtrec 1-800-424-9300



4300 Emperor Blvd - 400 Durham, NC 27703

Tel: 1-877-846-5393 Fax: 1-877-817-1716

[www.cancerdiagnostics.com](http://www.cancerdiagnostics.com)

**Made in USA**

MX1324 Acid Alcohol 1.0 l Gallons, 4 AW-25812 Rev.B 2020-06-17