

**COMPONENTS****CAS#**

Deionized Water	7732-18-5
Hydrochloric Acid	7647-01-0
EDTA	64-02-8

EMERGENCY CONTACT: Chemtrec 1-800-424-9300*For Invitro Diagnostic Use Only*

High Speed Bone Decalcifier

DehCL+™**DIRECTIONS FOR USE:**

Dehcl +™ is formulated to work faster than Dehcl™ (Original Formulation) and can be used on all types of calcified specimens including bone marrow samples. Fixed needle biopsies will decalcify in less than one hour. Sections will cut with ease and all H&E and special stains exhibit excellent stain detail.

DANGER: Contains dilute acid. May be corrosive to metallic parts, skin, eyes and clothing. Avoid prolonged contact. Harmful if swallowed. Harmful if inhaled. Causes severe skin burns and eye damage. Wash face, hands and any exposed skin thoroughly after handling. Do not eat, drink or smoke when using this product. Use only outdoors or in a well-ventilated area. Do not breathe dust/fume/gas/mist/vapors/spray. Wear protective gloves/protective clothing/eye protection/face protection.

IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Immediately call a POISON CENTER or doctor.

IF ON SKIN (OR HAIR): Remove/Take off immediately all contaminated clothing. Rinse skin with water/shower. Wash contaminated clothing before reuse.

IF INHALED: Remove victim to fresh air and keep at rest in a position comfortable for breathing. Call a POISON CENTER or doctor/physician if you feel unwell. Immediately call a POISON CENTER or doctor/physician.

IF SWALLOWED: Call a POISON CENTER or doctor/physician if you feel unwell. Rinse mouth. Do NOT induce vomiting.

IN CASE OF FIRE: Use CO₂, dry chemical, or foam for extinction.

www.cancerdiagnostics.com

Cancer Diagnostics, Inc. 4300 Emperor Blvd., 400 Durham, NC 27703 (877) 846 5393 Fax: (877) 817 1716

59°F (15°C) 86°F (30°C)

REF DP025 | Gal (3.8L) CUBE

Store locked up at room temperature.
Dispose of contents/container to an approved waste disposal plant.

MX1384 DehCL Plus CUBE AW-22803 Rev.B 2020-07-15

LOT



MADE IN USA