



RIBBON™

Tissue Section Adhesive

REF #PCM1005 | 1 Gallon (3.8L)

LOT



For In Vitro Diagnostics Use



Date Received: _____ Date Opened: _____

Directions: Use in the water bath to increase the affinity of the tissue for the slide. 5-10 milliliters are added to each liter of pre-warmed water in a tissue flotation bath. (0.5 to 1% ratio to water) Pre-mixed and ready-to-use, our Tissue Section Adhesive will not interfere with staining reactions and does not create background residue. Use in H&E Staining, special stains, and immunoperoxidase procedures.

NONHAZARDOUS CHEMICAL

This chemical is not considered hazardous by the 2012 OSHA Hazard Communication Standard (29 CFR 1910.1200)

Although this product does not normally require specific hazard precautions, chemical users should always take care to minimize personnel exposure and workplace contamination.

STORAGE & DISPOSAL

Store tightly closed at room temperature. Dispose of contents/container to an approved waste disposal plant.

NOTICE: Read Safety Data Sheet for additional information.

EMERGENCY CONTACT: Chemtrec 1-800-424-9300



CANCER
DIAGNOSTICS INC.

4300 Emperor Blvd - 400 Durham, NC 27703

Tel: 1-877-846-5393 Fax: 1-877-817-1716

www.cancerdiagnostics.com

Made in USA

MX1576 Ribbon TS Adhesive, 1 Gallon AW-5304794 Rev.C 2020-12-14